

► Mobile Commode Chair

► User manual

⚠ Safety instructions:

- ▶ The patient's body weight must not exceed 130 kg!
- ▶ The use of the commode chair may only be carried out with the assistance of a caretaker (nursing specialist)!
- ▶ The commode chair must only be used on a level and non-slippery surface!
- ▶ There is danger of jamming when the footrests and armrests are raised and lowered.
- ▶ Check the welding seams and screws regularly!
- ▶ The calf strap must be attached during transport of the patient!
- ▶ Always sit back in the chair and do not lean out of the chair, as there is risk of tipping!
- ▶ Do not use in the sauna or shower!
- ▶ The commode chair is not a toy! Take care of your children!
- ▶ Only remove the toilet bucket when nobody is sitting in the chair, danger of pinching!
- ▶ Ensure that the seat surface is firmly seated on the seat frame!
- ▶ The commode chair may only be used by persons who can control their balance while sitting.
- ▶ The commode chair or its footrests are not suitable for standing on them.
- ▶ If parts of the commode chair are bent or deformed, they must not be used!
- ▶ In case of serious incidents in connection with this product please contact the manufacturer or the competent authority.
- ▶ Only use original spare parts!

Warranty:

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly.


We reserve the right of modifications serving the technical improvement.

Mobile Commode Chair

Item No. 14000 / 2 brake rollers

Item No. 14002 / 4 brake rollers



 Sundo Homecare GmbH,
Bahnhofstraße 13, DE-30916 Isernhagen, www.sundo-homecare.de

 Valentine International Ltd., 8th Floor, No. 149, Sec 2, Ta Tung Road,
221 Hsichih District, New Taipei City, Taiwan R.O.C.

 Y. Sung Handelsvertretung, Duesselthaler Str. 24, DE-40211 Duesseldorf



sundo
HOMECARE



sundo
HOMECARE

► Mobile Commode Chair

Intended use:

The toilet chair is a medical device. It facilitates the toilet operation if it is not possible or difficult to use/reach the toilet. On the one hand, it can be used as a toilet replacement for the conventional purpose, but can also be placed over the toilet during the day as a support aid or used as a transport aid indoors.

Indication:

Difficulties in reaching / using the toilet in time due to different diseases (general weakness, severely limiting walking disabilities, urge or stress incontinence, faecal incontinence, chronic diarrhoea etc.), which requires a direct, spatial proximity of a toilet facility.

Contraindication:

Strong dizziness, disorientation, no balance/trunk control in sitting or similar requires the unconditional presence and assistance of a caregiver.

Versions:

There are different models/sizes, the selection should be made together with the specialist staff. The mobile toilet chair is available with two or four brake wheels. Selection, assembly and instruction in use may only be carried out by authorized, trained personnel. The toilet chair is designed for domestic use, may only come into contact with intact, uninjured skin and should not be used across patients. Any use other than the above is not intended. Sundo GmbH is not liable for any resulting damages, the risk is borne exclusively by the user.

Before each use / supplement:

First read the instructions for use completely and keep it until the end of the product life.

Check whether the commode chair ...tightened for first and every use.

The toilet chair is delivered pre-assembled and consists of frame, seat, cover plate and toilet bucket.

In case of damage, the product must not be used (any more), please contact your specialist dealer.



Assembly:

The Mobile Commode Chair is quickly assembled without tools in a few simple steps.

- Insert the backrest into the tubes on the underframe ① and tighten both screws.
- Snap the armrests into the brackets ②.
- Attach the footrests to the receiving pins and swivel them towards the center. The footrests must engage audibly.
- Mount the calf strap ⑤ on the footrest tube ④.

Before each use:

- Check that the backrest is correctly inserted, the screws are tightened and the armrests are properly locked.
- Check whether the parking brakes of the wheels work and the footrests are properly hooked in.

Use:

The removable, padded cover plate can be placed on the seat if the chair is used as a transport opportunity. To use the toilet equipment remove the cover plate, slide the toilet bucket **from behind without lid**. Now the chair is ready for use for the toilet aisle. Note: the edge of the bucket has a groove ("water edge"), which can be filled with clear water and contributes to the avoidance of odour after putting on the lid. After finishing the toilet procedure, proceed in reverse order and dispose of the contents of the bucket in the toilet. For use as a support aid, push the toilet chair without cover plate and without bucket backwards over the toilet, so that the cut-out is placed above the toilet. Determine the brakes. The toilet chair may only be used on flat and non-slip surfaces. For each change in or out of the toilet chair, you will need the help of a caregiver. Before changing, lock the brakes, fold the foot plates upwards and assist the patient in sitting down. Then fold down the foot plates again so that the patient can place his feet on it. Proceed to leave in reverse order. **Attention**, the use of the toilet chair as transport aid may only be done with the cover plate on!

Cover / seat / commode bucket supplement

- The black cover can be removed from the top. When reassembling, ensure that the locks fit into the white seat.
- The white seat is removed by pulling the front, rounded edge upwards. When fitting, at first it is hooked in at the back (backrest) and then pressed downwards.
- The toilet bucket can be pushed into the holder from behind (with or without cover).

Technical specifications:

Item No. 14000 / Mobile Commode Chair, 2 brake rollers			
Item No. 14002 / Mobile Commode Chair, 4 brake rollers			
Color:	grey/black	Sitzfläche:	approx. 45 x 44 cm
Total width:	approx. 56 cm	Seat height:	approx. 53 cm
Total height:	approx. 96 cm	max. load:	130 kg
Total depth:	approx. 92 cm	passable toilet height:	approx. 41 cm

Cleaning / Disinfection:

Do not use sharp or abrasive objects or detergents for cleaning. Use only mild detergents (for seat plate and bucket also plastic cleaners are possible) and soft wipes or sponges. Make sure that no water enters the pipes. All parts can be treated with commercially available chlorine-free disinfectants, bucket and seat plate should be disinfected after each use.

Attention, please observe the application rules of the manufacturers!