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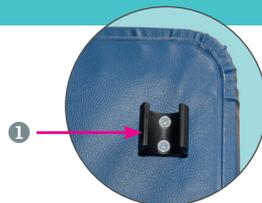
► Commode Chair, Height-Adjustable

Cleaning / Disinfection

- ▶ Do not use sharp or abrasive objects or detergents for cleaning. Use only mild, usual household cleaners (for seat plate and bucket also plastic cleaners are possible) and soft wipes or sponges. Make sure that no water enters the pipes. All parts can be treated with commercially available chlorine-free disinfectants, bucket and seat plate should be disinfected after each use.

- ▶ **Attention**, please observe the application rules of the manufacturers!
- ▶ For thorough cleaning/disinfection, the toilet seat can be removed by carefully loosening the two fastening clips ①. To reassemble the seat, reattach it to the rear tube with the clips.

⚠ Safety instructions

- ▶ The patient's body weight must not exceed 120 kg
- ▶ The Commode Chair must only be used on level and non-slip ground
- ▶ Always sit all the way back into the Commode Chair, do not lean far out, risk of tipping!
- ▶ Do not stand or climb on the chair!
- ▶ Do not use in the sauna or shower!
- ▶ The Commode Chair is no toy, take care of your children!
- ▶ The Commode Chair must only be used by people who can control their balance while sitting, otherwise a caregiver is required!
- ▶ In case of damage, the Commode Chair must not be used (any longer)!
- ▶ The Commode Chair must only come into contact with intact, uninjured skin
- ▶ Only remove the bucket when nobody is sitting in the chair, danger of pinching!
- ▶ When adjusting the height, make sure that all four legs are set to the same height and that the clips are safely locked in place!
- ▶ In case of serious incidents in connection with this product please contact the manufacturer or the competent authority.

Warranty

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly.

We reserve the right of modifications serving the technical improvement.

► User Manual

Commode Chair, Height-Adjustable

Item no. 14 200 / black-grey

Item no. 14 201 / blue

Distributed by:

Sundo Homecare GmbH, Bahnhofstraße 13, DE-30916 Isernhagen, www.sundo-homecare.de

Valentine International Ltd., 8th Floor, No. 149, Sec 2, Ta Tung Road, 221 Hsichih District, New Taipei City, Taiwan R.O.C.

EC REP Y. Sung Handelsvertretung, Duesselthaler Str. 24, 40211 Duesseldorf Germany

► Commode Chair, Height-Adjustable

Intended Use

The Commode Chair is a medical device. It facilitates the toilet operation if it is not possible or difficult to use/reach the toilet.

Indication

Difficulties in reaching / using the toilet in time due to different diseases (general weakness, severely limiting walking disabilities, urge or stress incontinence, faecal incontinence, chronic diarrhoea etc.), which requires a direct, spatial proximity of a toilet facility.

Contraindication

Strong dizziness, disorientation, no balance/trunk control in sitting or similar requires the unconditional presence and assistance of a caregiver.

Versions / Description

There are different models/sizes, the selection should be made together with the specialist staff. The Commode Chair can be used as a toilet replacement for the conventional purpose, but can also be placed over the toilet during the day as a support aid or used as a sitting aid. The height adjustability of the legs enables an adaptation to different body sizes and conditions.

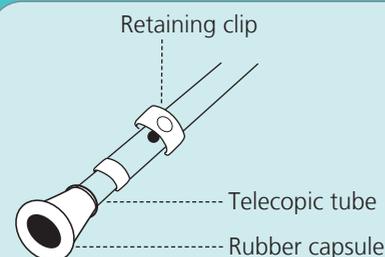
Selection, assembly and instruction in use must only be carried out by authorized, trained personnel. The toilet chair is designed for domestic use, must only come into contact with intact, uninjured skin and must not be used across patients. Any use other than the above is not intended. Sundo GmbH or the manufacturer are not liable for any resulting damages, the risk is borne exclusively by the user.

Before first use /Maintenance

First, read the instructions for use completely and keep it until the end of the product life. Check that the Commode Chair is complete and undamaged. It is delivered pre-assembled and consists of a frame, seat, cover plate and toilet bucket. In case of damage, the aid must not be used (any longer), please contact your specialist dealer. No further maintenance is required. The Commode Chair is quickly constructed with just a few simple steps, without tools.

Height adjustment

The height of the Commode Chair is adjusted on the legs. First the holding clip must be pulled out, then the telescopic tube is pulled out to the desired length. The holding clip is put back on and has to snap in completely (see picture).

- Pull out the retaining clip
- Pull out the telescopic tube to the desired height
- Let the retaining clip snap in safely

The retaining clip must be attached to the chair tube and the pins must emerge from the hole visibly and noticeably

Use

The removable, padded cover plate can be placed on the seat when the chair is used as a seat. To use the toilet equipment, remove the cover plate, fold up the toilet seat and mount the toilet bucket from above or from the side. Remove the lid and fold down the seat. Now the chair is ready for use for the toilet aisle. **Note:** the edge of the bucket has a groove ("water edge") that can be filled with clear water and contributes to the avoidance of odour after the lid is placed. After completing the toilet aisle, proceed in reverse order and dispose of the contents of the bucket in the toilet. For use as a support aid, adjust the Commode Chair with the help of height adjustment so that it can be pushed over the toilet. Then (without bucket) push over the toilet so that the seat cutout is placed over the toilet.

Technical data

Item no. 14 200 / Commode Chair, Height-Adjustable, black-grey/ Item no. 14 201 / Commode Chair, Height-Adjustable, blue			
Material:	steel pipe, powder-coated toilet seat / cover plate: foam / PVC	Max. load:	120 kg
Seat width:	appr. 48 cm	Weight:	cpl. 7,95 kg
Seat height:	sevenfold adjustable from appr. 43-59 cm (incl. seat)	Lifespan:	appr. 10 years
Total height:	appr. 78 - 92,5 cm	Total width:	57 / 59 cm