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## INSTRUCTIONS MANUAL EN

### IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a mobility aid. He or she will advise and inform you about the most appropriate use in your case.

Distributor: this leaflet must be given to the users of this product.

### PRODUCT DESTINATION

The wheelchair movflight are aids mobility for people with reduced mobility. These products are intended for use indoors and outdoors and can be folded up for easy storage or transport.

### PRODUCT DESCRIPTION

1. Handgrip
2. Backrest Upholstery
3. Armrest
4. Seat Upholstery
5. Legrest
6. Leg Support

7. Footplate
8. Front Wheel
9. Front Caster
10. Brake
11. Handgrip
12. Rear Wheel

### SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 120 Kg.
- Make sure that all parts are secure and in good working order.
- Do not use the product on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- Do not make any adjustments to the product while it is in use.
- Check that the brakes work properly before use.
- During use, lock the brakes for all actions.
- All wheels must always be in contact with the ground.
- When the product is in the stationary position, the brakes must be locked. Do not attempt to move the product if the brakes are locked.
- Make sure that the footrests are properly engaged.
- Make sure that the disassembly pin is securely engaged in the rear wheels and frame of the chair.
- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the walker on a wet, icy, or snowy slope.
- When using the product, do not exceed a speed of 3-5 km/h (3-5 mph).
- Do not stand on the product's footrests as there is a risk of the product tipping over.
- Do not use the product when wheel tire is out of air or has not enough air. Pay attention to air pressure : the air pressure when inflating tires and air pressure must be restricted in 3.5 kg/cm<sup>2</sup> as to the rear wheel.
- If any parts are damaged, broken, worn or missing, contact the dealer.

### MOUNTING

#### 1. Unfolding

Hold the handles with both hands to separate the frame slightly. Push down both sides of the seat upholstery to flatness lightly by hand, so that the wheelchair has been unfolded. (Figure 1).

Note: when unfolding, do not pull the frame hardly for fear of damaging the parts. Don't hold the seat tubes with fingers to avoid injuring your fingers when pushing down the seat upholstery.

#### 2. Folding

First turn over both the footplates, lift up both sides of the seat upholstery lightly with hands, and then it can be folded. (Figure 2)

#### 3. Adjust the footplate

First loosen the locknut with the spanner, pull down the slide tube and adjust the footplate to a proper position, and then lock and fix the bolts with the spanner in a clockwise direction. (Figure 3).

#### 4. Method of promoting the wheelchair by oneself

The user sits on the wheelchair stably, and then holds the hand rims of the rear wheels with both hands. At the same time promote the wheelchair by a balanced force with both hands to make it move forward, release the hands. When the wheelchair slow down and needs to be operated once again after traveling for some distance, repeat these steps as mentioned above. (Figure 6)

## USE

### 1. Use of the brakes

The user holds the brake handles with both hands when the wheelchair stops, and then moves the brake handles forward to self-locking condition, to make the rear wheels stop completely to keep still. The process for braking is completed. (Figure 5)

### 2. Operation method of cross the obstacles

In case of coming across with obstacles, the accompany should cinch the handle with both hands, trample the end of tube of the wheelchair, lean back the wheelchair to lift the front castor, then cross the obstacles. (Figure 6)

### 3. The knocked-down method of flip back armrest (fixed armrest is excluded)

When detaching the armrest, first press the armrest fastener with one hand to separate it from the front bracket of seat tube, then you can flip it back. When assembling the armrest

fastener aim at the position of the front bracket of the seat tube and press lightly to self-locking fastener. (Figure 7)

### 4. The knocked-down method of detachable legrest (fixed legrest is excluded)

When assembling the legrest, you should first aim the up and down positioning holes at the up and down fixed pins, hang the legrest and then circumvolve the legrest to the spring from outside to inside to reach a state of self-locking; when detaching the legrest, you should first, then turn the legrest outside into a certain angle, lift it to get out of the fixed pins, and take the legrest at last. (Figure 8)

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth to avoid rustiness.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use. Check the brakes by pushing the handles into the locked position, then trying to push the product.
- Examine Wheelchairs to see whether all parts and bolts are loose or not after usage in some period
- of time. If they are loose, people involved should adjust them in time

for fear of insecure hidden trouble

- Seat and back upholstery shall be kept clean and tidy so as to protect users' skin.
- If there is a malfunction, do not repair the product yourself. Consult an authorized dealer.

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Observe the conditions for storing and stowing your product :
  - In a dry and temperate place
  - Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
  - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
  - All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## TECHNICAL SPECIFICATIONS

Overall dimensions	Width. 63 x Depth. 103 x Height 91 cm
Seat dimensions	Width 45 x Depth. 40 x height 50 cm
Seat Width	46 cm
Seat Depth	40 cm
Floor to seat Height	50 cm
Backrest Height	41 cm
Folded Width	26.5 cm
Front Caster Size	20 cm
Rear Caster Size	61 cm
Structure	Steel grey colour
Weight	18.5 kg
Maximum weight supported	120 kg

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by

the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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